

2C

Maintain

Quick start guide SAM 350P AED

Watch the video


Maintain
Your AED



heartsine.com/videos

Weekly


Check that status indicator
is flashing green*

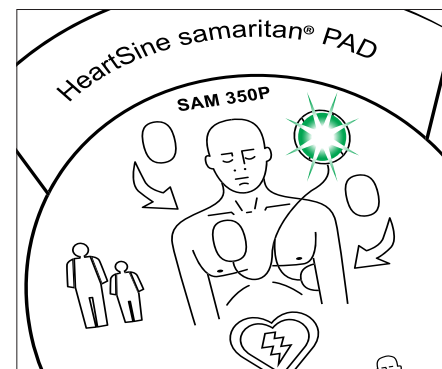


* If not, see Troubleshooting in
User Handbook.

After use

1. Remove pads from
person and stick together

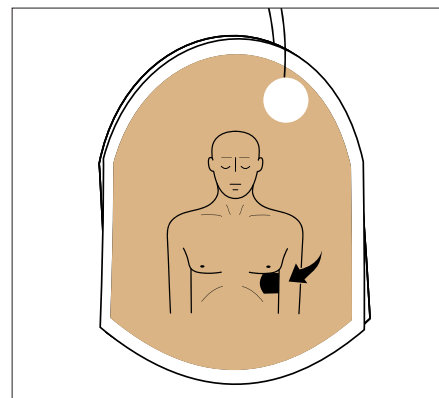

2. Remove entire Pad-Pak
from AED

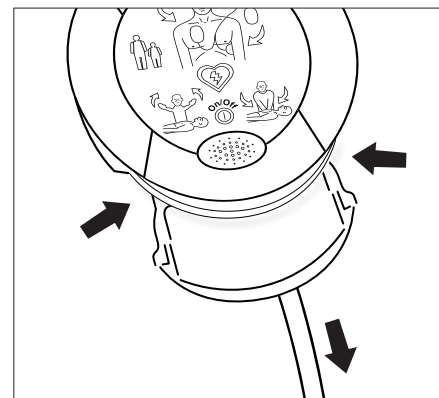


Monthly


Check for damage




Check expiration date;
replace if expired

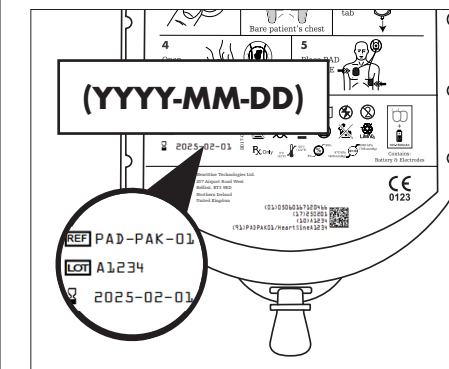


3. Cut cords to separate pads from battery cartridge

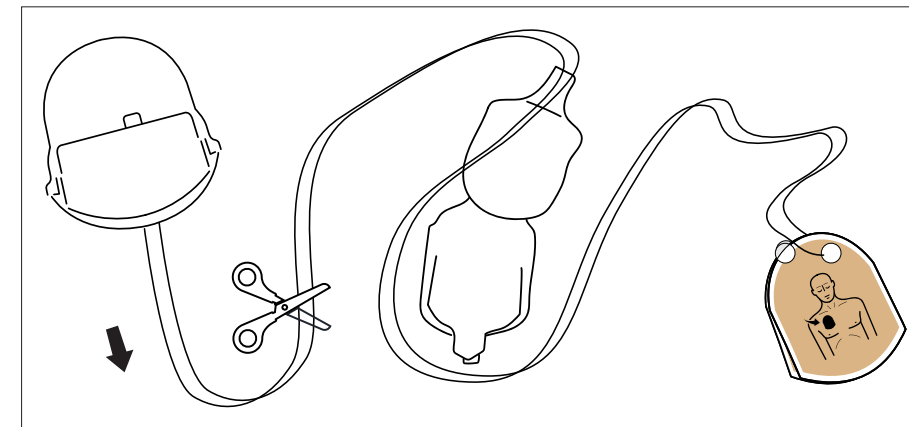


4. Dispose of electrode pads and battery cartridge separately

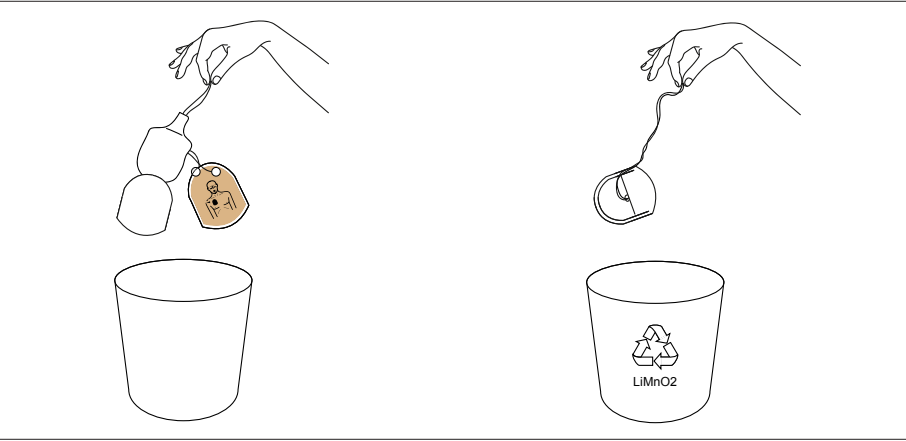


7. Insert new Pad-Pak and note expiration date in User Handbook

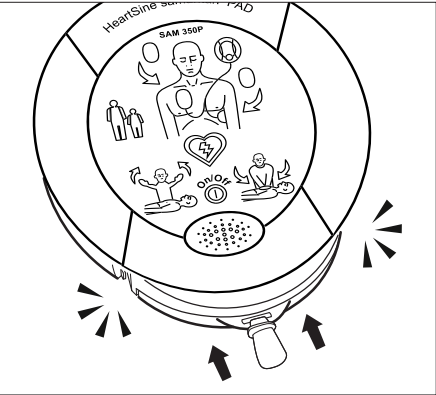

8. Check that status indicator is flashing green and store your AED

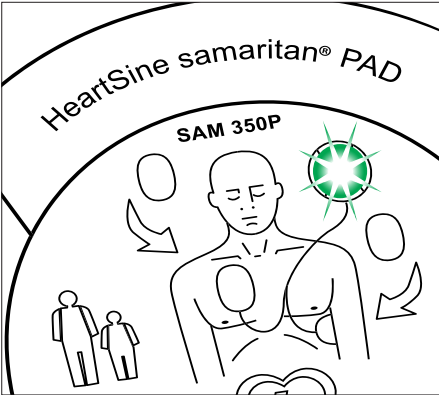


5. Wipe AED with damp cloth

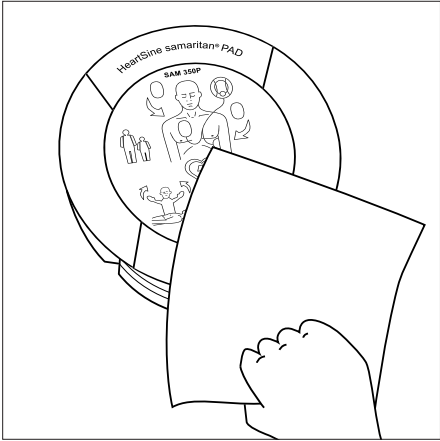

6. Check for damage



9. Tell us when you use your AED!

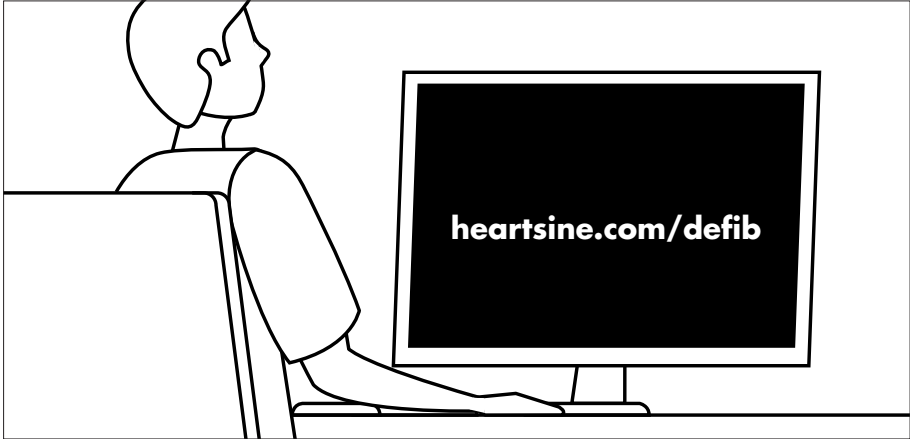

Please watch videos:
heartsine.com/videos

heartsine.com

For detailed instructions, please read the User Handbook included with your AED.

For further information contact us at heartsinesupport@stryker.com or visit our website at heartsine.com

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: HeartSine, Pad-Pak, Pediatric-Pak, samaritan, Stryker. All other trademarks are trademarks of their respective owners or holders.

The absence of a product, feature, or service name, or logo from this list does not constitute a waiver of Stryker's trademark or other intellectual property rights concerning that name or logo.

Please report any serious incident that occurs with this device to HeartSine Technologies, Ltd. and to the U.S. Food and Drug Administration as per local regulations.

CE 0123 Pad-Pak; Pediatric-Pak

HeartSine samaritan PAD: UL Classified. See complete marking on product.

Date of Issue: 02/2022
Made in U.K.
M0000004326-AB EN-US
© 2022 HeartSine Technologies. All rights reserved.



HeartSine Technologies Ltd.
207 Airport Road West
Belfast
Northern Ireland
BT3 9ED
United Kingdom
Tel +44 28 9093 9400
Fax +44 28 9093 9401
heartsinesupport@stryker.com
heartsine.com

Distributed in U.S. by:

Stryker Emergency Care
11811 Willows Road NE
Redmond, WA, 98052 U.S.A.
Toll free 800 442 1142
strykeremergencycare.com