

2B

Use


Quick start guide

SAM 350P AED




For use on a young child, see User Handbook and video: Use Your AED on a Child


1. Press On/Off and follow voice directions

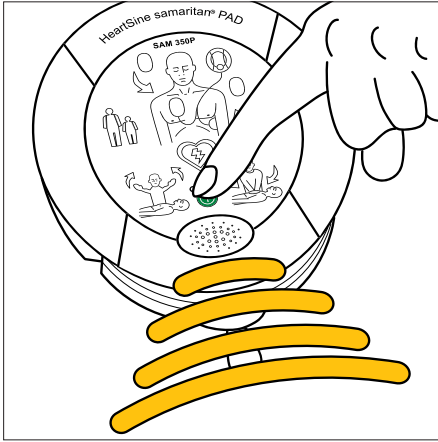


2. Expose and dry chest area

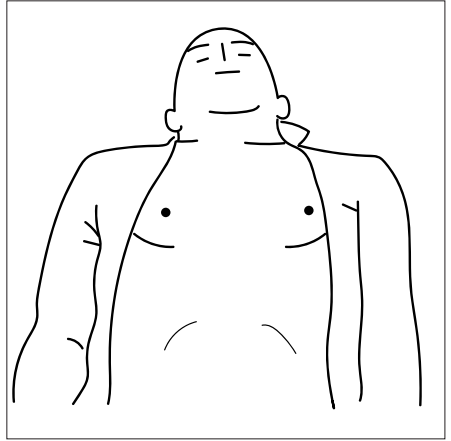


5. Peel first pad from liner

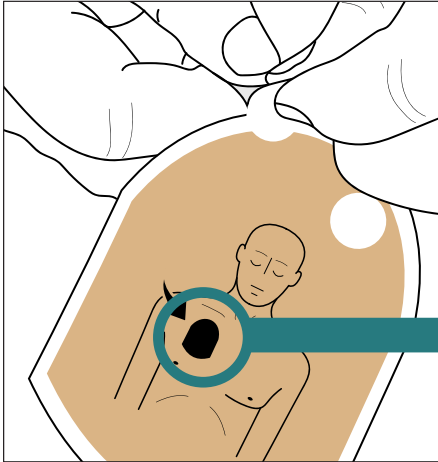


6. Place first pad

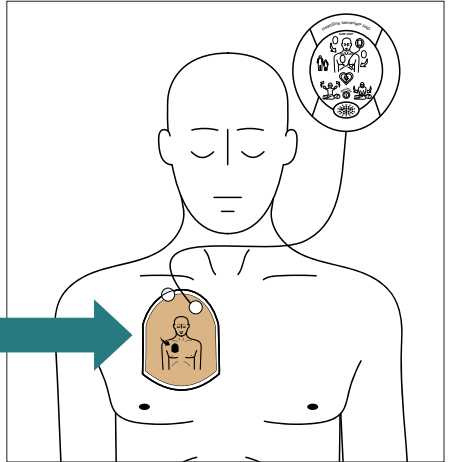


3. Pull green tab to remove pads

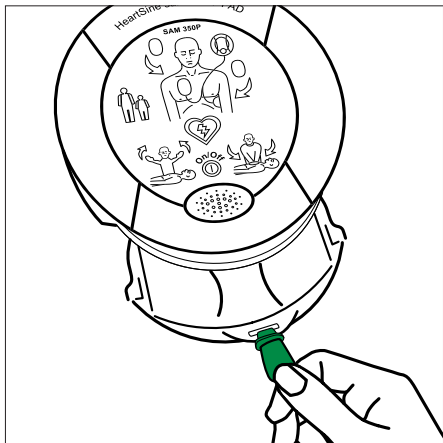


4. Open electrode pouch pads

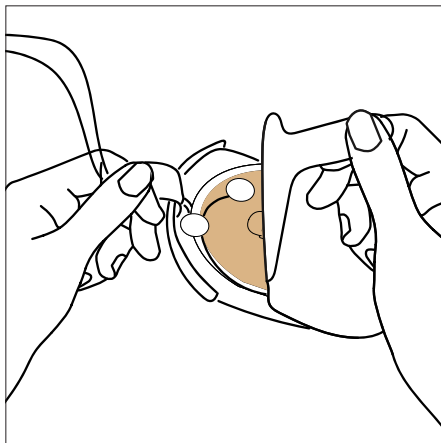


7. Peel second pad from liner

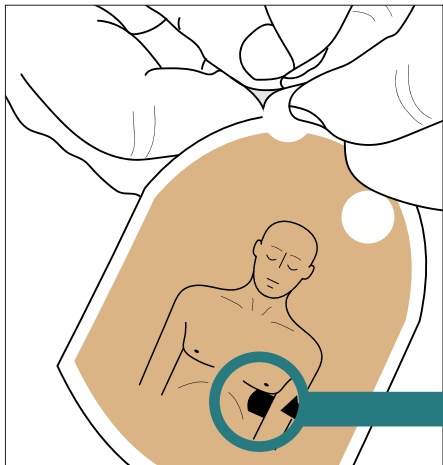


8. Place second pad

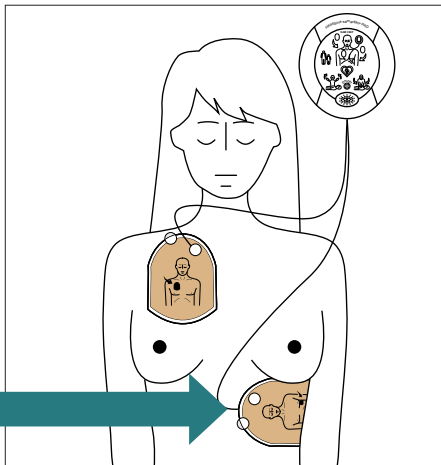


9. Do not touch person

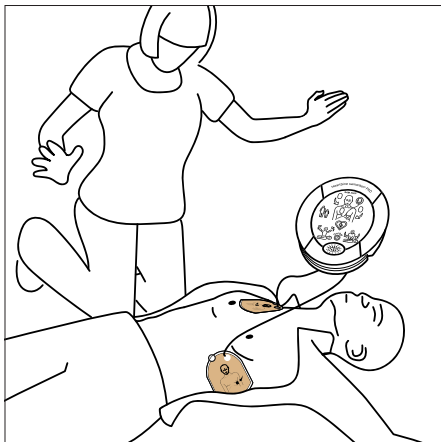


10. If AED says “Shock advised”, press shock button **if told to do so**

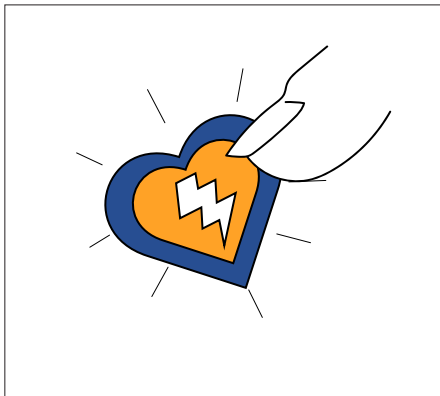


12. Follow voice directions until help arrives

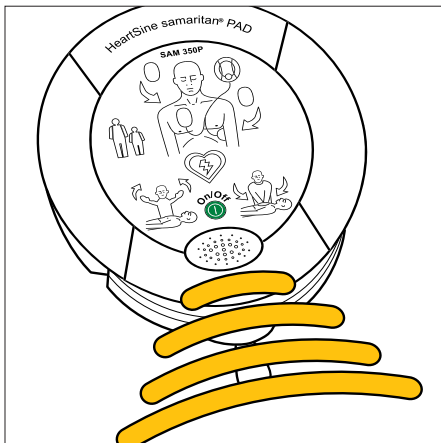

13. Press On/Off to turn off AED

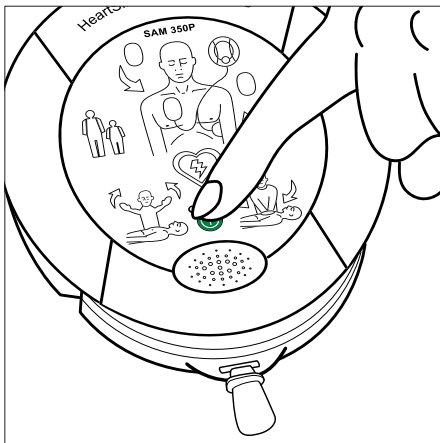

11. Begin CPR when directed. Push hard at least 2 inches deep.
Push fast in time with the beat.

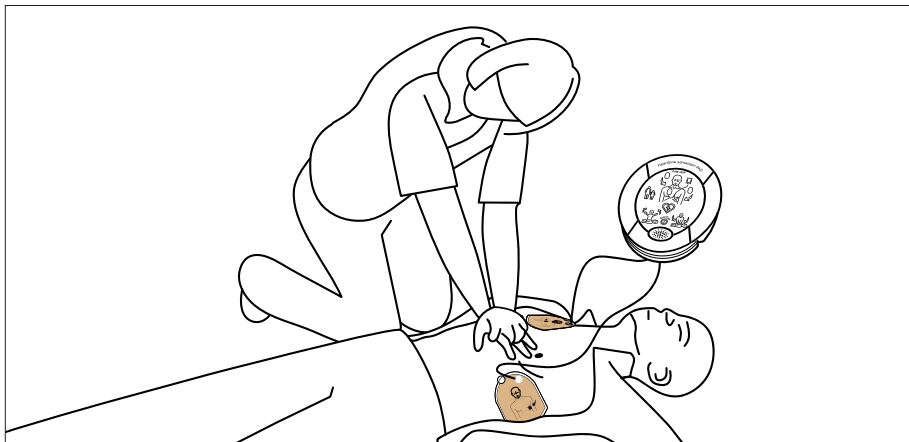

14. For disposal of pads after use, see Maintain guide

15. Watch these videos to be ready to use your AED

HeartSine

2C

Maintain

Quick start guide
SAM 350P AED

Use Your AED on an Adult
Use Your AED on a Child



[heart sine.com/videos](https://www.heart sine.com/videos)

Please watch videos: heartsine.com/videos

heartsine.com

For detailed instructions, please read the User Handbook included with your AED.

For further information contact us at heartsinesupport@stryker.com or visit our website at heartsine.com

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: HeartSine, Pad-Pak, Pediatric-Pak, samaritan, Stryker. All other trademarks are trademarks of their respective owners or holders.

The absence of a product, feature, or service name, or logo from this list does not constitute a waiver of Stryker's trademark or other intellectual property rights concerning that name or logo.

Please report any serious incident that occurs with this device to HeartSine Technologies, Ltd. and to the U.S. Food and Drug Administration as per local regulations.

 **0123** Pad-Pak; Pediatric-Pak

 HeartSine samaritan PAD: UL Classified. See complete marking on product.

Date of Issue: **01/2022**

Made in U.K.

M0000004327-AB EN-US

© 2022 HeartSine Technologies. All rights reserved.



HeartSine Technologies Ltd.
207 Airport Road West
Belfast
Northern Ireland
BT3 9ED
United Kingdom
Tel +44 28 9093 9400
Fax +44 28 9093 9401
heartsinesupport@stryker.com
heartsine.com

Distributed in U.S. by:

Stryker Emergency Care
11811 Willows Road NE
Redmond, WA, 98052 U.S.A.
Toll free 800 442 1142
strykeremergencycare.com